Slovenský

Návod na použitie

Dni cyklu

1 2 3 4 5 6 7 8 9 10 11 12 13 14 15 16 17 18 19 20 21 22 23 24 25 26 27

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plodné dni

určite neplodné dni

Prefert® Vaginálny gél

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Quick start to use Prefert®:

1. Insert applicator into vagina, 5 min
2. Apply gel to the applicator tip
3. Insert applicator into vagina
4. Apply gel to the applicator tip
5. Apply gel to the applicator tip
6. Apply gel to the applicator tip

Prefert® Vaginal Gel
Lubricant when trying to conceive

Note: Please read instructions before use!

Mode of action: Prefert® promotes sperm activity at the time of ovulation:

- Provides an optimal environment for sperm
- Ideal pH for natural conception
- Improves mobility of sperm in comparison to classical lubricants
- Reduces the osmotic barrier for sperm
- Improves osmolarity

When to use: For temporary use on the 3 most fertile days of the female cycle (time of ovulation).

Use only once within 24 hours.

Directions: Introduce 5 minutes before sexual intercourse.

1. Prior to intercourse, take out a disposable syringe of Prefert® Vaginal Gel and the reusable applicator.
2. Take the gray cap off the syringe.
3. Attach the applicator to the syringe. Insert the vaginal applicator into the vagina.
4. Apply pressure to the plunger of the syringe to push the Prefert® Vaginal Gel into the vagina. Remove the syringe with the applicator.
5. Rinse out the applicator and store it for the next application.
6. Now intercourse can begin. The contents of the package are for 4 applications.

Discard the empty syringe in household waste after use.

Clean and rinse the vaginal applicator with warm water. Can be reused.

Free of parabens.
Clinically tested in Germany.

Warnings: Prefert® vaginal is not a contraceptive gel. Prefert® does not kill Sperm cells or interfere with sperm function. If the use of Prefert® vaginal gel causes irritations or discomfort, stop the use of this product and, if necessary, ask your healthcare provider. Keep this product out of reach of children. If you have had recently any vaginal surgery, ask your healthcare provider before use. Do not use Prefert® vaginal gel, if you have an active vaginal infection.

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