Vaginal applicator

for gel or cream

for M9
thread

Medintim

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Vaginal applicator for gel or cream

Instruction for use:

Thoroughly wash your hands.

1. Fill the applicator with the gel or cream. Screw the applicator onto the tube and carefully squeeze the gel or cream out of the tube until the indicated dosage mark is reached.
2. Remove the applicator from the tube.
3. Insert the applicator as deeply as possible into the vagina. This is most simply performed lying down with the legs slightly raised. In this position, the pelvic muscles are relaxed.
4. Once you have correctly positioned the applicator, empty it by gently pressing down on the plunger.
5. Remove the empty applicator from the vagina without pulling on the plunger, as some of the gel or cream would otherwise be sucked back into the applicator.
6. Once used, dispose of single-use applicators in the household waste.

Important information:

- Do not use gel or cream applicators during pregnancy. There is a risk of injury if handled incorrectly, as the area around the cervix is very sensitive in pregnant women. Ask your doctor for more information.
- The applicator fits tubes with 9 mm thread, e. g. Caya® Gel or Contragel® green.

Gebrauchsanweisung beachten / Read instructions for use / Consulter la notice / Läs bruksanvisningen / Følg bruksanvisningen / Noudata käyttöohjeita / Brugsanvisningen skal overholdes / Consulte las instrucciones de uso / Observar as instruções de utilização / Consultare le istruzioni per l'uso / Olvassa el a használati útmutatót / Gebruiksaanwijzing raadplegen / Dodržujte návod k použití / Przestrzegać instrukcji użycia / Upoštevajte navodila za uporabo / Dodržiavajte návod na použitie /

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KESSEL medintim GmbH